

# AccuSet<sup>™</sup> HCV Verification Panel 2

0810-0304 / Batch #10670482

## OVERVIEW


AccuSet<sup>™</sup> HCV Verification Panel 2, 0810-0304 / Batch #10670482, is a 20-member validation panel of undiluted, naturally occurring plasma samples (1 vial per member, 1.0 mL per vial). Panel members represent bleeds from multiple individuals positive for HCV. Each sample represents a single collection event. No preservatives were added.

Test results from commercially available HCV assays are included for characterization of the panel members. This panel of human plasma samples demonstrates a range of anti-HCV reactivity.

For Research Use Only. Not for use in diagnostic procedures. Data are offered for informational purposes. LGC/SeraCare Life Sciences does not claim that others can duplicate test results exactly.

CAUTION: Potentially infectious materials. Follow Universal Precautions. Some panel members were found positive for HCV; all were found negative for anti-HIV-1/2 and HBsAg. This does not ensure the absence of these or other human pathogens.

AccuSet<sup>™</sup> HCV Verification Panel 2

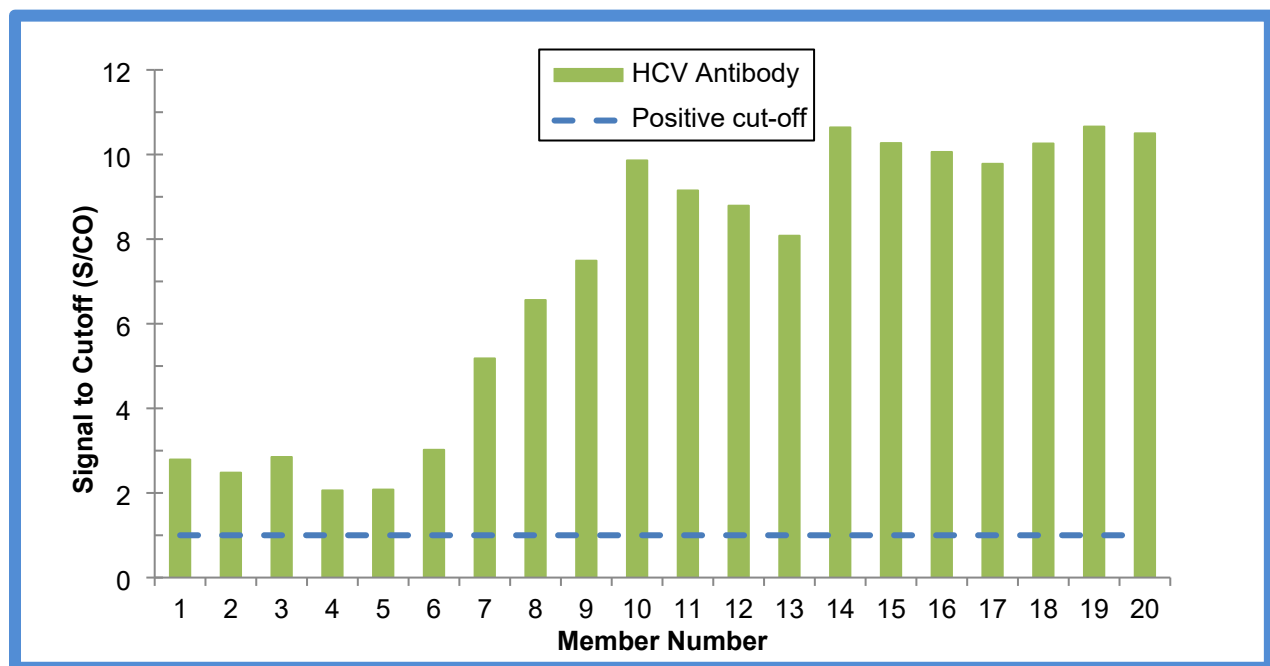

*This graph demonstrates reactivity amongst panel members from the Abbott Alinity s (Anti-HCV) test method.*

AccuSet™ HCV  
 Verification Panel 2  
 0810-0304 / Batch #10670482

**Panel Member Information**

Panel Member	SeraCare Batch #	SeraCare Donor ID #	Bleed Date
01	9115962	N-19538-043	05-May-2000
02	9242321	NA	18-Aug-2009
03	BM217249	0800270029	19-Jan-2007
04	9219994	6010582914	28-Sep-2007
05	9145128	24048445	19-May-2002
06	9146125	27042822	27-Nov-2002
07	9143706	23042803	16-Jul-2002
08	BM216787	0010932285	30-Aug-2006
09	9144479	23043712	24-Jul-2002
10	BM217193	0331067518	15-Dec-2006
11	BM202058	N05904520	12-Mar-2003
12	9154287	27056005	20-Jul-2003
13	9144480	23042776	16-Jul-2002
14	9146116	27036813	03-Sep-2002
15	9147502	27044317	20-Dec-2002
16	9146115	27040848	30-Oct-2002
17	9146118	27040795	29-Oct-2002
18	9147493	27044102	18-Dec-2002
19	9146120	27039326	09-Oct-2002
20	9147492	27044050	17-Dec-2002

NA = Not Available

# AccuSet™ HCV Verification Panel 2

0810-0304 / Batch #10670482

## HCV Antibody and HCV Antigen<sup>1</sup>

Panel Member	Abbott ARCHITECT Anti-HCV (S/CO)	Abbott Alinity s Anti-HCV (S/CO)	Abbott Alinity s Anti-HCV II (S/CO)	Roche cobas® Elecsys Anti-HCV II (COI)	Abbott ARCHITECT HCV Ag (fmol/L)
01	<b>3.97</b>	<b>2.79</b>	<b>128.12</b>	<b>199.95</b>	0.00
02	<b>3.07</b>	<b>2.48</b>	<b>64.71</b>	<b>51.07</b>	0.00
03	<b>3.68</b>	<b>2.85</b>	<b>97.80</b>	<b>117.35</b>	0.00
04	<b>3.25</b>	<b>2.06</b>	<b>92.81</b>	<b>84.74</b>	0.00
05	<b>3.47</b>	<b>2.08</b>	<b>98.70</b>	<b>107.95</b>	0.00
06	<b>5.26</b>	<b>3.02</b>	<b>46.90</b>	<b>28.73</b>	<b>145.67</b>
07	<b>7.24</b>	<b>5.18</b>	<b>128.53</b>	<b>184.50</b>	0.00
08	<b>9.23</b>	<b>6.56</b>	<b>33.32</b>	<b>145.80</b>	<b>111.65</b>
09	<b>10.41</b>	<b>7.49</b>	<b>143.48</b>	<b>255.05</b>	0.00
10	<b>10.89</b>	<b>9.86</b>	<b>97.69</b>	<b>39.09</b>	<b>2919.98</b>
11	<b>10.71</b>	<b>9.15</b>	<b>28.12</b>	<b>32.23</b>	<b>10.61</b>
12	<b>11.17</b>	<b>8.79</b>	<b>33.27</b>	<b>110.50</b>	0.00
13	<b>11.96</b>	<b>8.08</b>	<b>111.14</b>	<b>84.35</b>	0.00
14	<b>13.02</b>	<b>10.64</b>	<b>41.44</b>	<b>29.84</b>	<b>3031.94</b>
15	<b>11.00</b>	<b>10.27</b>	<b>98.83</b>	<b>30.01</b>	<b>9819.37</b>
16	<b>14.22</b>	<b>10.06</b>	<b>74.73</b>	<b>46.72</b>	<b>190.39</b>
17	<b>13.53</b>	<b>9.78</b>	<b>57.26</b>	<b>40.00</b>	<b>290.05</b>
18	<b>15.02</b>	<b>10.26</b>	<b>65.89</b>	<b>40.87</b>	<b>12.40</b>
19	<b>14.52</b>	<b>10.66</b>	<b>40.96</b>	<b>25.54</b>	<b>43.04</b>
20	<b>14.96</b>	<b>10.50</b>	<b>22.87</b>	<b>14.31</b>	0.00
Test Date	11-Jul-2023	07-Jul-2023	18-Oct-2024	01-Aug-2023	19-Jul-2023
Test Site	LGC	RL	RL	RL	RL
Kit Part Code	1L79	NA	NA	NA	NA
Kit Lot No.	50482BE00	50274BE00	62454BE00	NA	51301LP76
Kit Exp. Date	28-Jan-2024	09-Feb-2024	28-Apr-2025	NA	02-Mar-2024

<sup>1</sup>Results are reported as the mean result of duplicate testing; positive/reactive results are noted in bold red.  
RL = Reference Lab; LGC = LGC Clinical Diagnostics; NA = Not Available

# AccuSet™ HCV

## Verification Panel 2

0810-0304 / Batch #10670482

### HCV Confirmatory Assay

#### INNOGENETICS® INNO-LIA™ HCV Score<sup>1</sup>

#### Band Results

Panel Member	HCV Antibody Interpretation	C1	C2	E2	NS3	NS4	NS5
01	<b>POS</b>	1+	1+	-	2+	-	-
02	<b>POS</b>	2+	1+	-	2+	-	-
03	<b>POS</b>	1+	1+	<u>±</u>	1+	-	-
04	IND	-	-	-	1+	-	-
05	<b>POS</b>	-	-	-	1+	1+	-
06	<b>POS</b>	4+	2+	-	-	2+	3+
07	<b>POS</b>	1+	1+	-	2+	1+	-
08	<b>POS</b>	2+	2+	-	2+	-	1+
09	<b>POS</b>	4+	1+	-	3+	2+	1+
10	<b>POS</b>	2+	2+	2+	4+	-	-
11	<b>POS</b>	3+	2+	1+	4+	4+	4+
12	<b>POS</b>	3+	2+	-	3+	1+	-
13	<b>POS</b>	-	-	-	3+	3+	-
14	<b>POS</b>	3+	2+	1+	4+	4+	4+
15	<b>POS</b>	3+	2+	1+	4+	-	-
16	<b>POS</b>	3+	3+	3+	4+	3+	1+
17	<b>POS</b>	1+	2+	1+	4+	1+	-
18	<b>POS</b>	4+	4+	2+	4+	2+	2+
19	<b>POS</b>	3+	3+	2+	4+	2+	3+
20	<b>POS</b>	4+	2+	-	4+	4+	3+

Test Date 19-Jul-2023

Test Site RL

Kit Part Code NA

Kit Lot No. 410791

Kit Exp. Date 30-Apr-2024

<sup>1</sup>Results are reported as singlicate testing; positive/reactive results are noted in bold red.

RL = Reference Lab; NA = Not Available

POS = Positive; IND = Indeterminate

The package insert for this panel can be found at [www.seracare.com](http://www.seracare.com).

A printed copy of the package insert or data sheet may be requested by email at [CDx-Info@LGCGroup.com](mailto:CDx-Info@LGCGroup.com) or by phone at 508.244.6400.